

BTS

100 liter
Cat #: 540/100
Net Contents: 4.0 kg (100 liter)

SHORT-TERM BOAR SEMEN EXTENDER

PREPARATION: Dissolve 40.0g per 1 liter sterile distilled water at 32-38°C. For optimal antibiotic efficacy, do not prepare more than 24 hours prior to use. Maintain at 35°C for 30 minutes to allow for pH stabilization before extending semen.

APPLICATION: Following preparation, slowly dilute fresh semen with BTS to a final concentration of 2-5 billion sperm per dose. For extended storage cool semen immediately to 16-17°C and store at a constant temperature for up to 48 hours.

QUALITY CONTROL: Warm semen to 37°C for a minimum of 20 minutes prior to motility evaluation.

STORAGE: Store unopened BTS at or below 22°C. Refrigerate and use within 72 hours after adding purified water.

WARNING: Keep out of reach of children. Not for human or animal consumption or drug or clinical use. Not for injection.

6064 McKee Rd
Fitchburg WI 53719 USA
(877) 270-1230



www.genepro-inc.com

MADE IN THE USA