

CHLORHEXI LUBE

"CHLORHEXI LUBE" is an all purpose non-spermicidal lubricating gel. Designed to lubricate hands, gloves, arms, and sleeves while performing obstetrical work, vaginal examinations, and rectal examinations.

"CHLORHEXI LUBE" may also be used as a lubricant for catheters and stomach tubes.

- Non-Irritating
- Non Spermicidal
- Water Soluble

WARNING:
Contact with eyes may cause irritation and temporary discomfort.

FIRST AID
EYES: Rinse with plenty of water. Remove contacts if present. Continue for at least 5 minutes. Obtain medical attention if irritation persists.

SKIN: Wash off with soap and water. No irritation should occur as a result from contact with this product under normal conditions.

INHALATION: Not applicable

INGESTION: May cause stomach irritation. Contact Physician if irritation persists.

FLAMMABLE: This product is non-flammable.

STORAGE: Keep away from high heat and freezing temperatures. Keep container tightly closed when not in use.

FOR ANIMAL USE ONLY
KEEP OUT OF REACH OF CHILDREN



Manufactured For:

Net Contents: One Gallon (3.785 L) (4 gallons / case)

BEFORE USE, READ SDS FOR ALL SAFETY PRECAUTIONS AND PROPER USAGE

USAGE:

For use as a lubricant on hands, arms and instruments when performing obstetrical, vaginal and rectal examinations.

Used to lubricate and minimize irritation to user's skin and animal's mucous membranes, for animal use only.

DIRECTIONS:

Intrauterine: Add approximately four ounces of CHLORHEXI LUBE to warm water and mix well.

Apply CHLORHEXI LUBE to gloves, sleeves or instruments when performing vaginal and rectal examinations. After use, rinse thoroughly with water to remove excess from exposed areas of animal.

24 HOUR
EMERGENCY RESPONSE NUMBER
1-800-424-9300



Rev. 10/5/2015 T