


SIZE: 3.5"x 3.5"

5/C 1- SPOT WHITE 2-YELLOW 3-MAGENTA 4-CYAN 5-BLACK DIELINE

JOB#172428-DATE:08/14/18-PART#Preserv EXT607

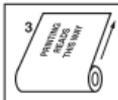

Preserv[®] xtra



**CONCENTRATED EXTENDER
FOR BOAR SEMEN DILUTION**

Storage: Store unopened extender at or below 22°C. Store opened extender in a tightly sealed container at or below 8°C. After adding purified water, store below 8°C and use within 72 hours.

Preparation: Dissolve 40 grams per liter of purified water at 32-38°C. For optimal performance do not prepare more than 24 hours in advance of use. Dissolve product completely before use.

Warning: Keep out of reach of children. Not for human or animal consumption or drug or clinical use.

Application: Dilute fresh semen with extender to a final concentration of <70 million sperm per mL. Store extended semen at a constant temperature of 15-17°C.

Quality Control: Warm semen to 37°C for a minimum of 20 minutes prior to motility evaluation or evaluate the semen on a caffeine coated slide.

Ingredients: Available upon request. Contains antibiotics as a preservative.

**EXT 607
Made in the USA**



6064 McKee Road, Fitchburg, WI USA
Tel: (608) 270-1230
www.genepro-inc.com

**FLEXO-
GRAPHICS**
LLC

APPROVED

DATE:

SIGN: